

ZPad™

A Recommended Protocol




STABILIZATION DEVICE & CATHETER CUSHION

AFTER CATHETER INSERTION, APPLY SKIN PREP AND ALLOW TO DRY THOROUGHLY.

- Remove blue cover liner



- Press screw (luer-lock) on blue landing pad with edge of luer-lock against ledge

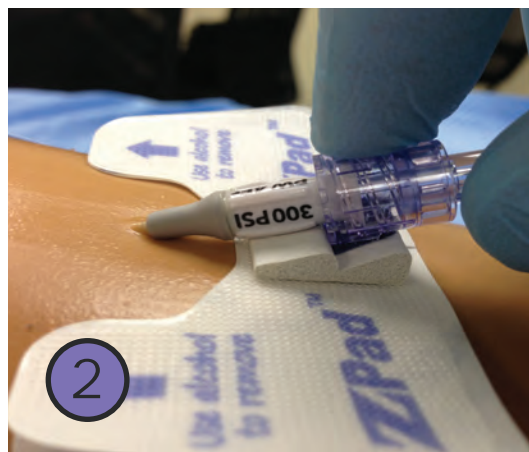


- PEEL away liner. Place Z-Pad on skin




APPLY CHLORHEXIDINE PATCH AT CATHETER INSERTION SITE.


- Apply tape over hub (Optional)

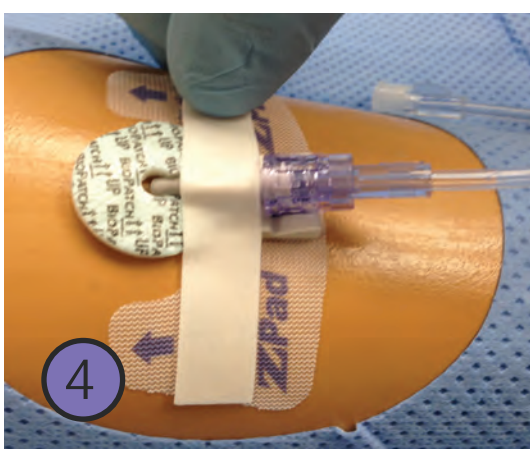

- Chevron second tape strip (Optional)

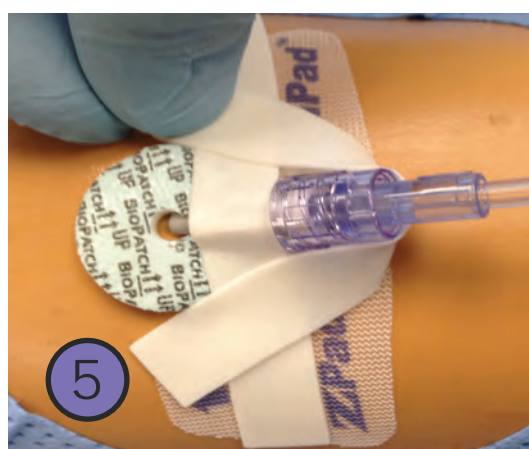

- Cover with transparent dressing

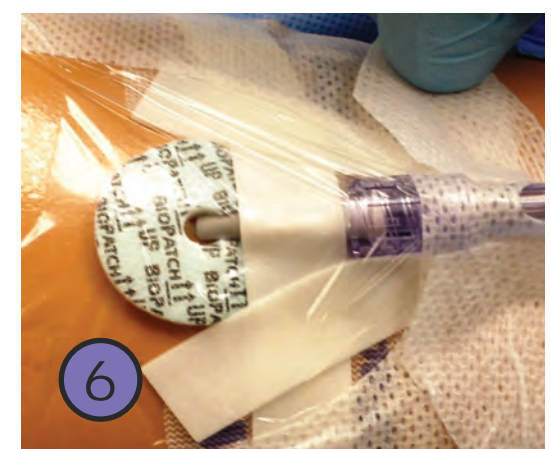

Z-Pad Removal (Change Z-pad every 7 days or sooner if indicated)

A. Stretch-remove transparent dressing.

B. Stretch-remove overlying tape strips.

BE CAREFUL TO AVOID CAUSING CATHETER DISPLACEMENT

C. Use alcohol to gently remove Z-Pad

D. Separate luer-lock from Z-Pad and replace with new Z-Pad

REPLACE TUBING ACCORDING TO INSTITUTION POLICY & PROCEDURE